


Instructions on how to export a CBCT scan from Invivo software in .dcm format:


1. Open the CBCT scan you wish to export with Invivo Anatomage software.
2. Click on **“Save As”** (not *Save Project As*).

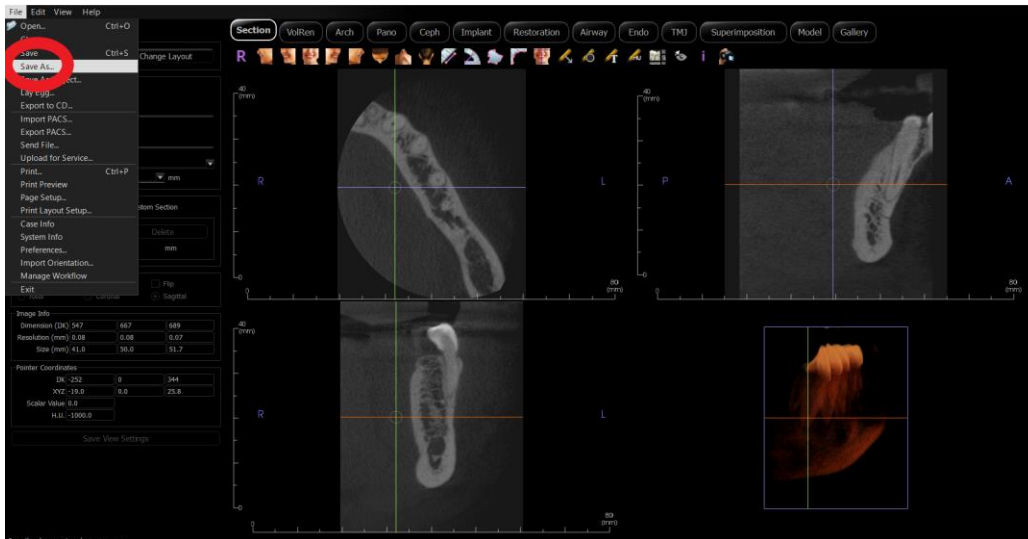


3. Choose **“Multi-file DICOM with .dcm extension”** and click **OK**.



4. Click on **“Create a New Folder”** on the Desktop and rename it with the patient’s name. Choose it as the location to save the scan. Click OK.

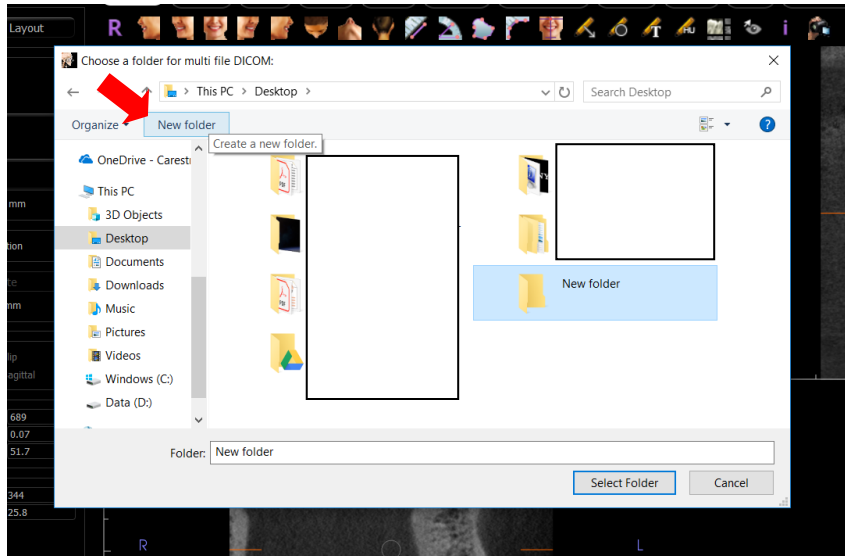

5. Once the export is finished, **right-click** on the folder on your Desktop that has the patient's name on it, go to **"Send to"**, and choose **"Compressed (zipped) folder"**. This will create a new icon on your Desktop that will have the patient's name on it but the icon will most likely be a folder with a zipped on it. That is the file we need.

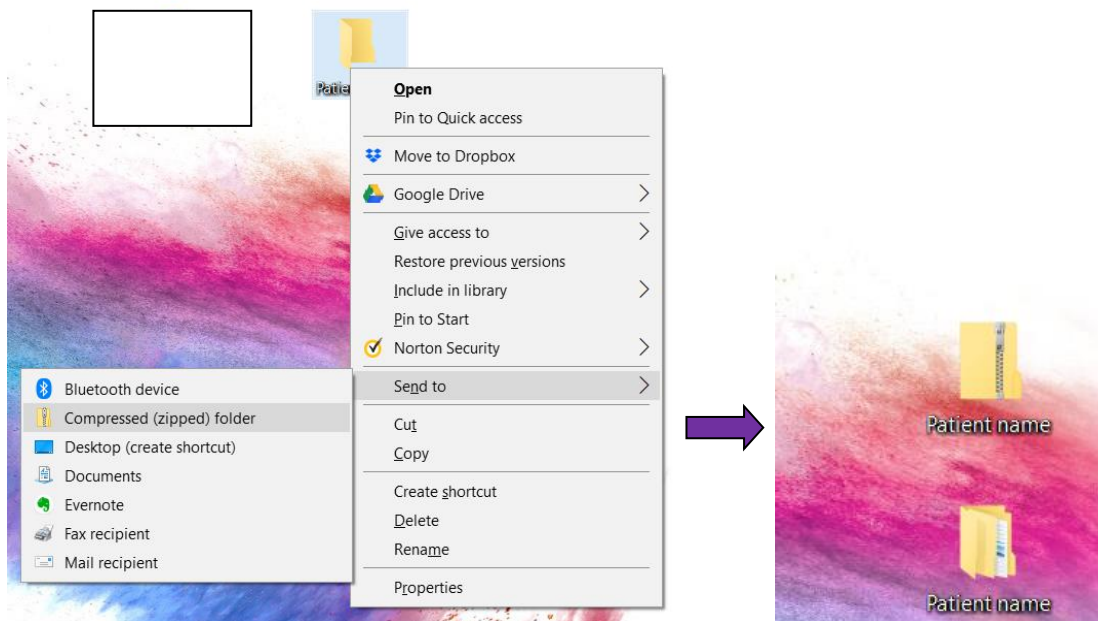

6. Upload the **compressed (zipped) folder** to the HIPAA compliant web portal with the required clinical information.