



Instructions on how to export a CBCT scan with EZ3D-i Vatech:

1. Open the scan with the **EZ3D-i** software.
2. On the side toolbar, click on **“Export”** under main menu.

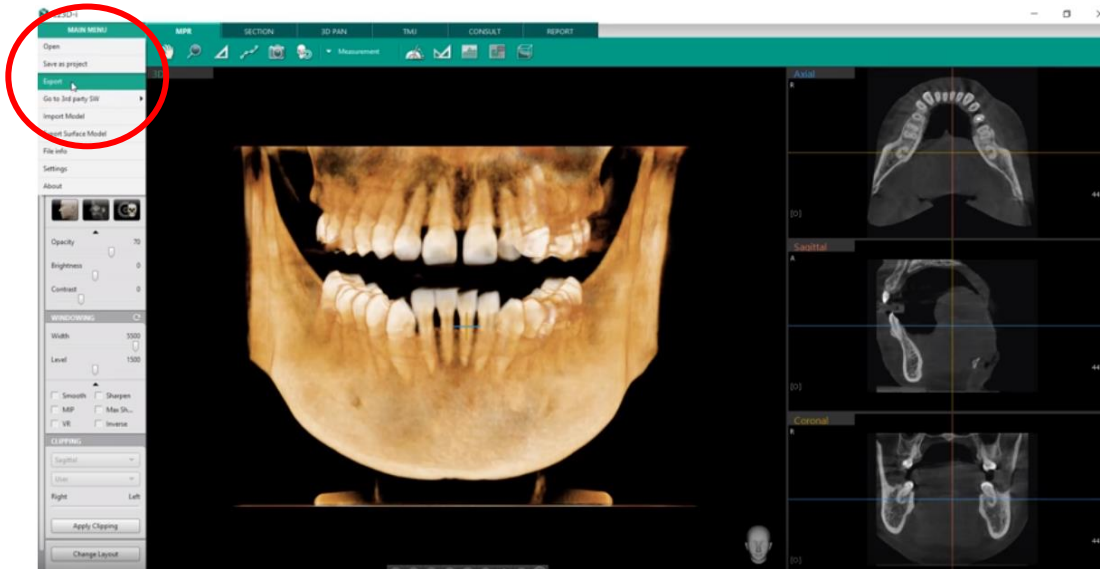


3. When the Export pop-up appears, select **“Local disk”** for save type and **Lossless** for compression.

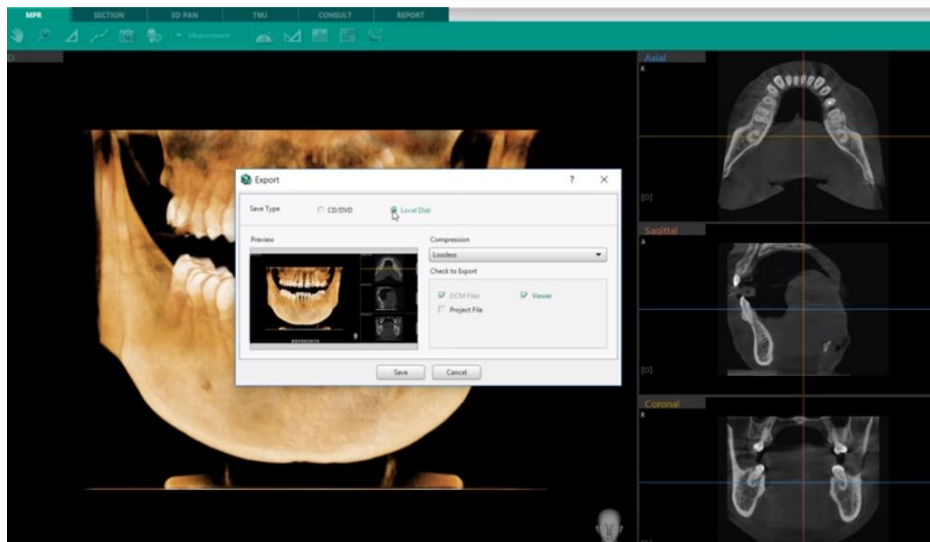

4. Browse to the Desktop and create a new folder with the patient's name on it and select that folder.
5. The DICOM files will now be exported.

6. Once the export is finished, **right-click** on the folder on your Desktop that has the patient's name on it, go to "**Send to**", and choose "**Compressed (zipped) folder**". This will create a new icon on your Desktop that will have the patient's name on it but the icon will most likely be a folder with a zipper on it. That is the file we need.

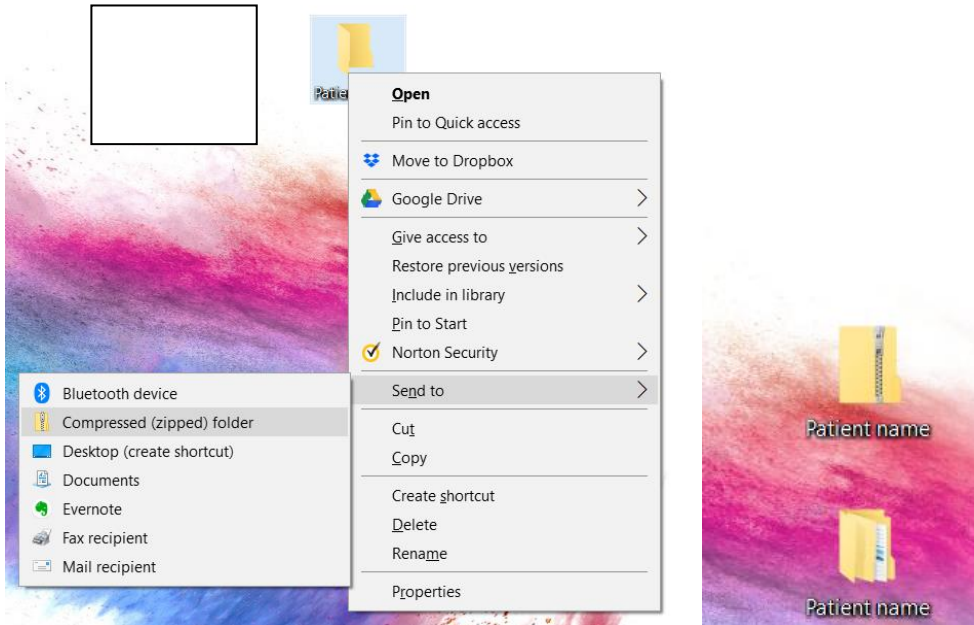

7. Upload the **compressed (zipped) folder** to the HIPAA compliant web portal with the required clinical information.