


## Export a CBCT scan from CS 3D Imaging Software


**Step 1:** Open the scan you want to export with the CS 3D Imaging Software.




**Step 2:** Click on the icon “**Export the volume**” in the upper horizontal tab.

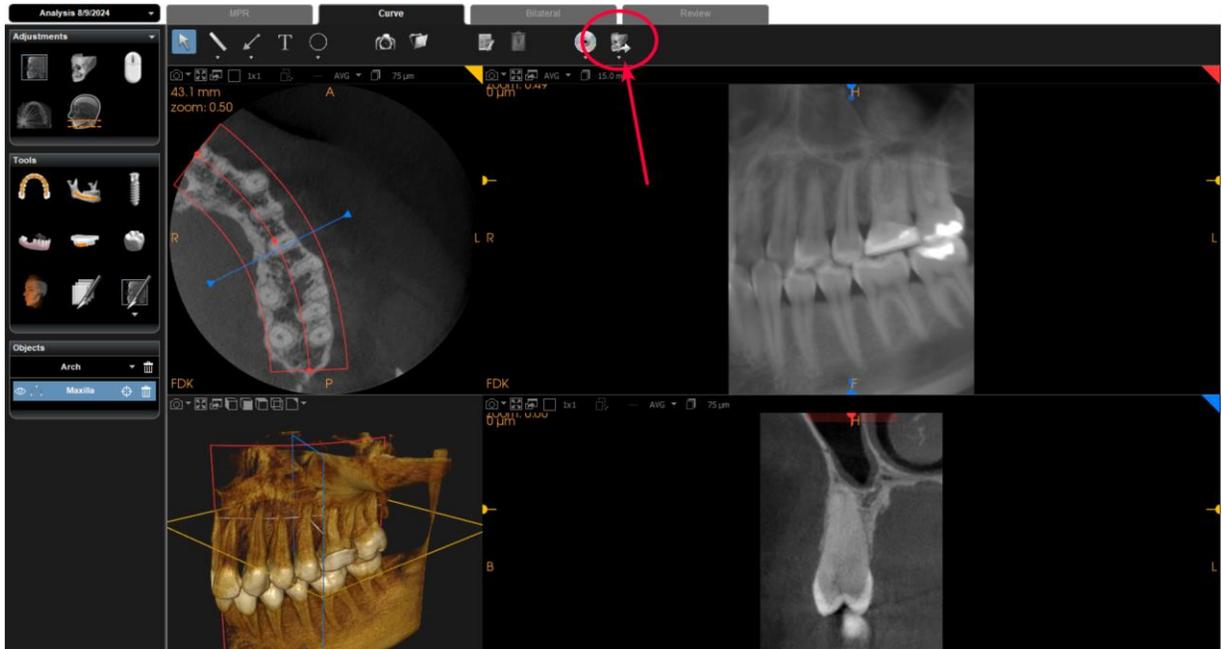

**Step 3:** A “**volume export**” tab will pop up.

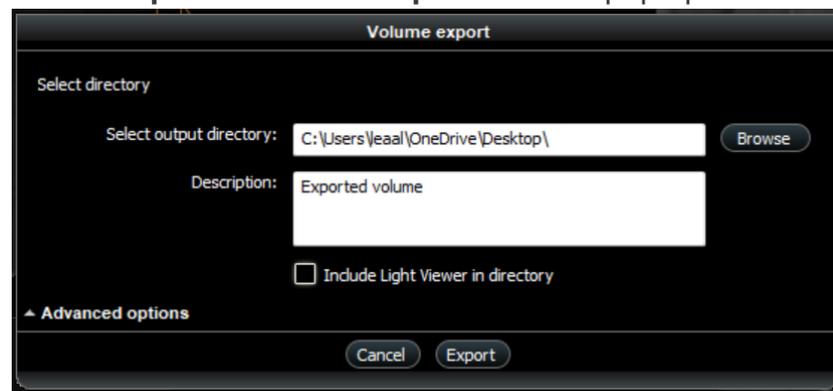


**Step 4:** Select output directory - click on **“Browse”**. Select **“Desktop”**. Click on **“Make New Folder”** and change it with the patient’s name.

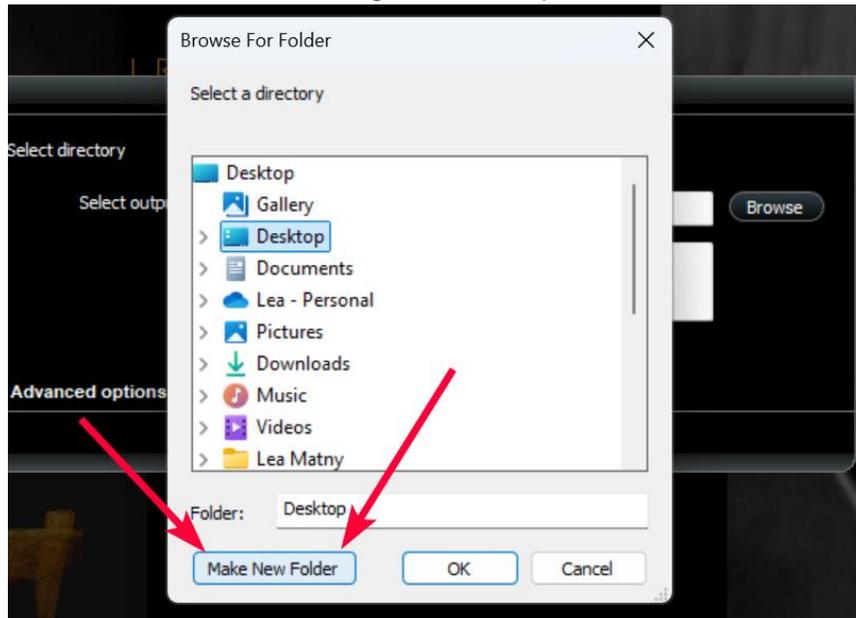



*Note: The output directory is where the folder will be saved.*

**Step 5:** Click on **Export**.

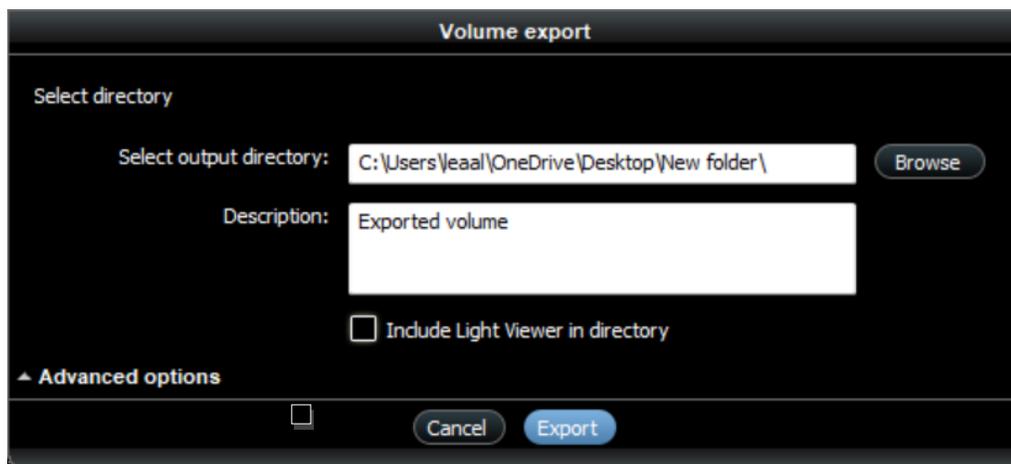

**Step 6:** Go to the desktop. Find the folder you just exported. Right-click on the folder. Select **“Send to”** → **“Compressed (zipped) folder”**.

*This will create a duplicate folder in the same location, but with a zipper icon. This is the file we need.  
(images below)*

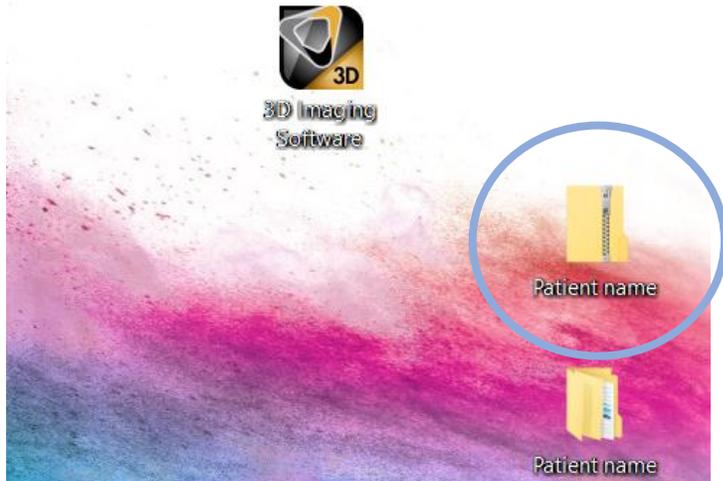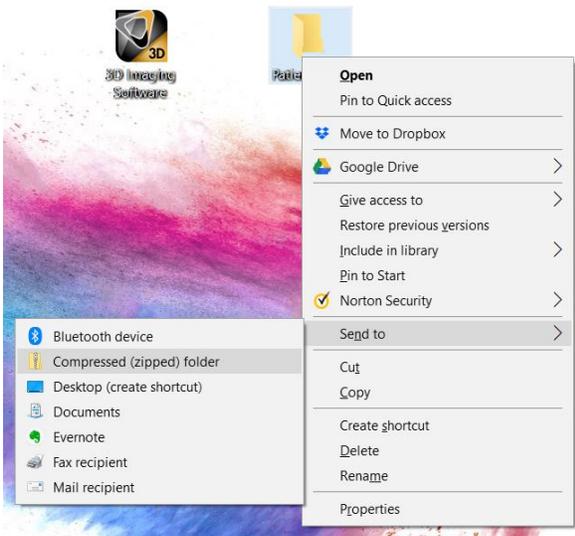

If you are using mysecurepractice, please click on ['Create a case'](#) to fill in patient's information.

If you are using hightail, please click on ['hightail upload'](#) to fill in patient's information.