



## Export CBCT scan from J-Morita (i-Dixel) - 1


1. On your desktop, create a new folder with the patient's name.
2. Open the patient's folder in the i-Dixel software.
3. Under "Tools", click on "Export", then choose "Save as file".

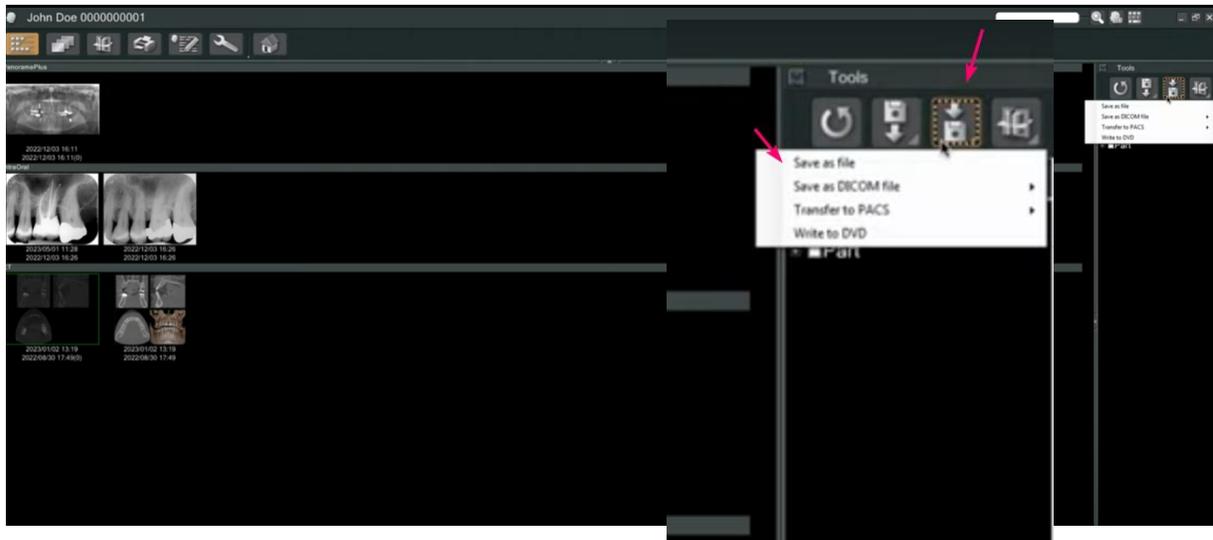


4. The following "Export CT data" box will open. Make sure you select "CT data".

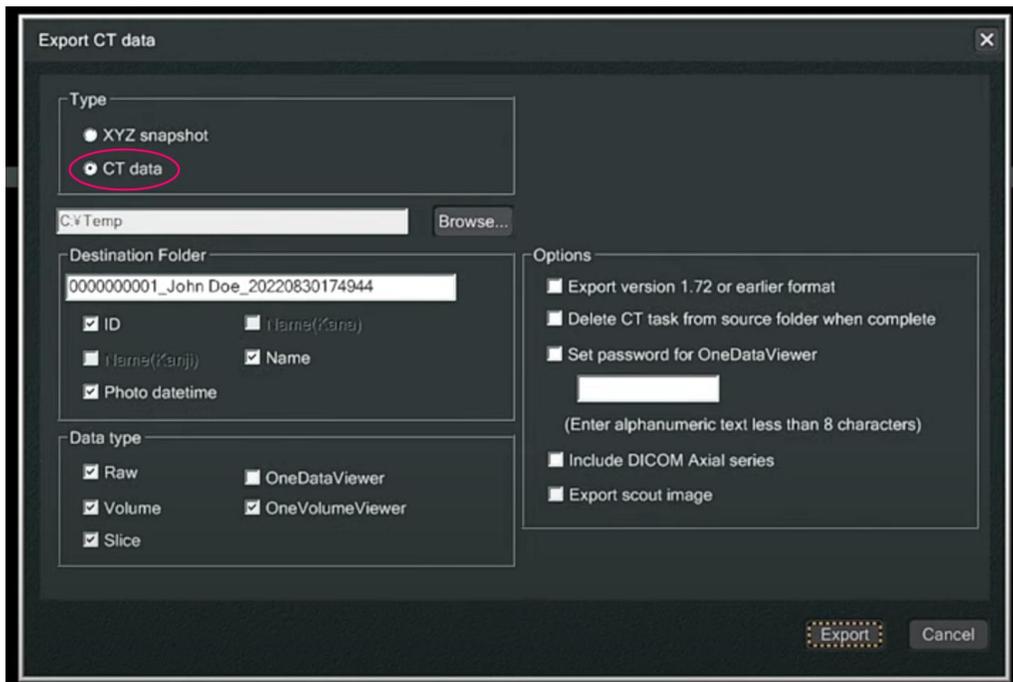

- Then, click on browse. Select the folder you created on desktop as destination folder

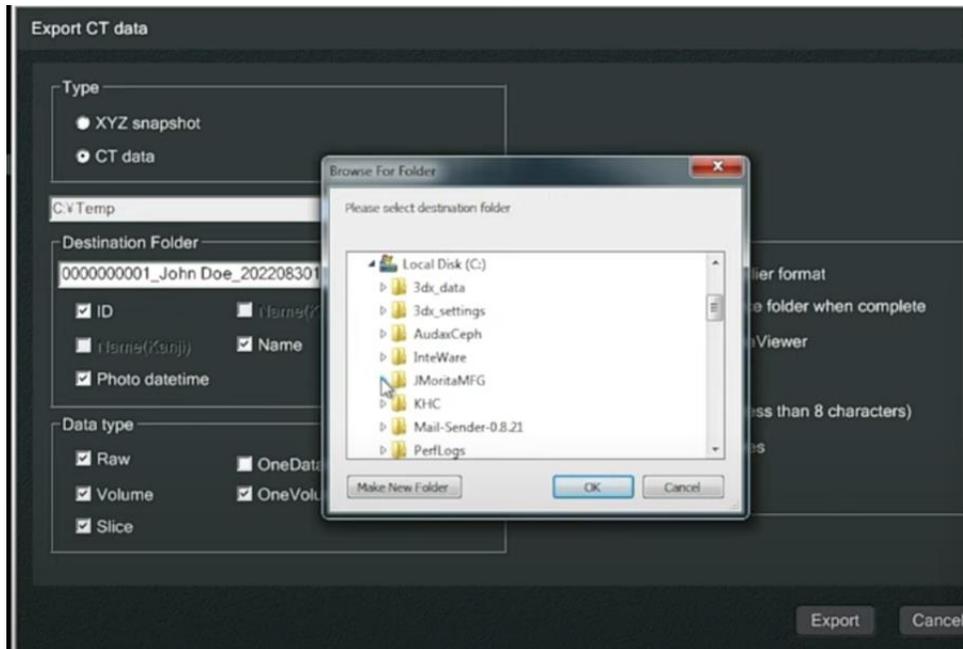

- Go to your desktop, find the folder you created on the desktop.
- Right-click on the folder. Select "**Send to**" and choose "**Compressed (zipped) folder**". A new zipped folder with the same name will be created in the same location. Please make sure you upload the zipped folder with the ZIPPER icon.

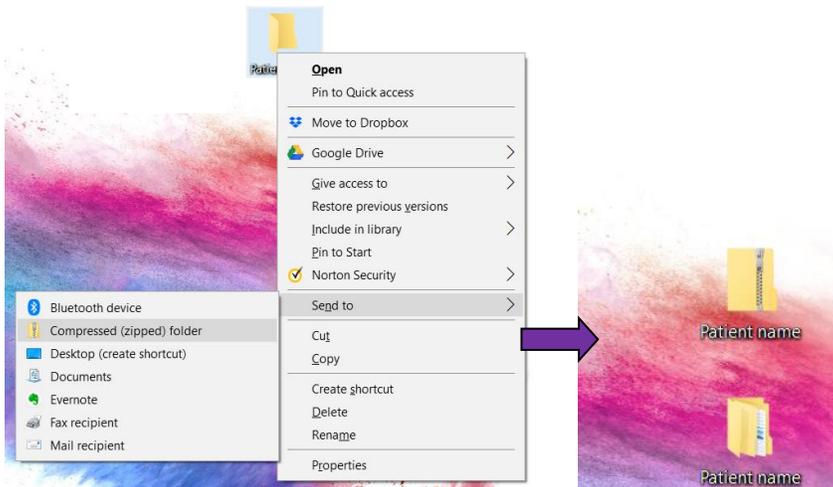

[Click here for instructions to create a case with mysecurepractice.](#)

[Click here for instructions to create a case with Hightail.](#)