



### Export CBCT scan from Planmeca Romexis

1. On your desktop, create a new folder with the patient’s name.
2. Open CBCT scan with the Planmeca Romexis software.
3. In the top toolbar, click on “Export volume”.

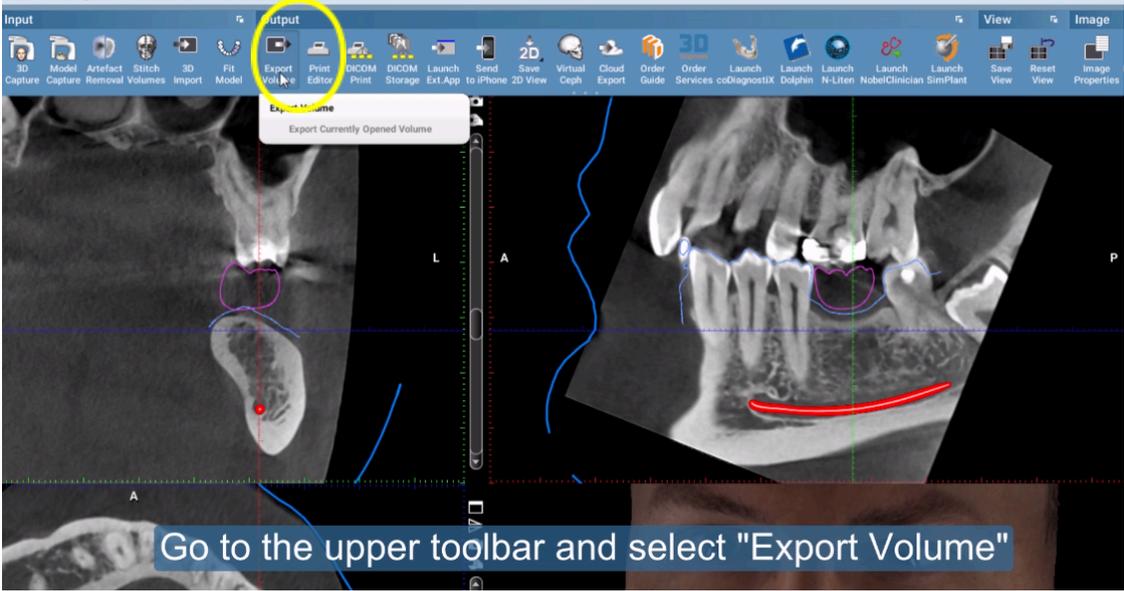


4. Under “File & Format options”, select “Set of Single Frame DICOM files”.

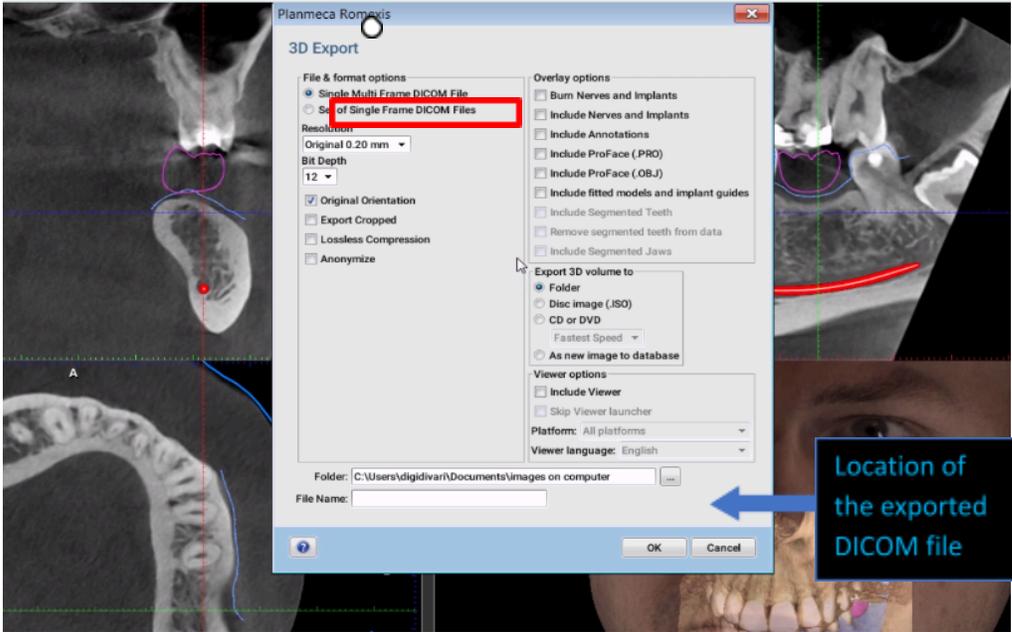



Make sure settings are same as below.

- a. **File & Format Options:** Set of Single Frame DICOM Files
  - b. **Resolution:** Original
  - c. **'Anonymize':** Unchecked
  - d. **Overlay options:** Unchecked
  - e. **Export 3D Volume to:** Folder
  - f. **Viewer Options:** Unchecked
  - g. **Folder:** Click on "...". Find the folder you created on the Desktop.
5. Click OK to export.
6. Go to your desktop, find the folder you created on the desktop.
7. Right-click on the folder. Select "**Send to**" and choose "**Compressed (zipped) folder**".  
A new zipped folder with the same name will be created in the same location. Please make sure you upload the zipped folder with the ZIPPER icon.

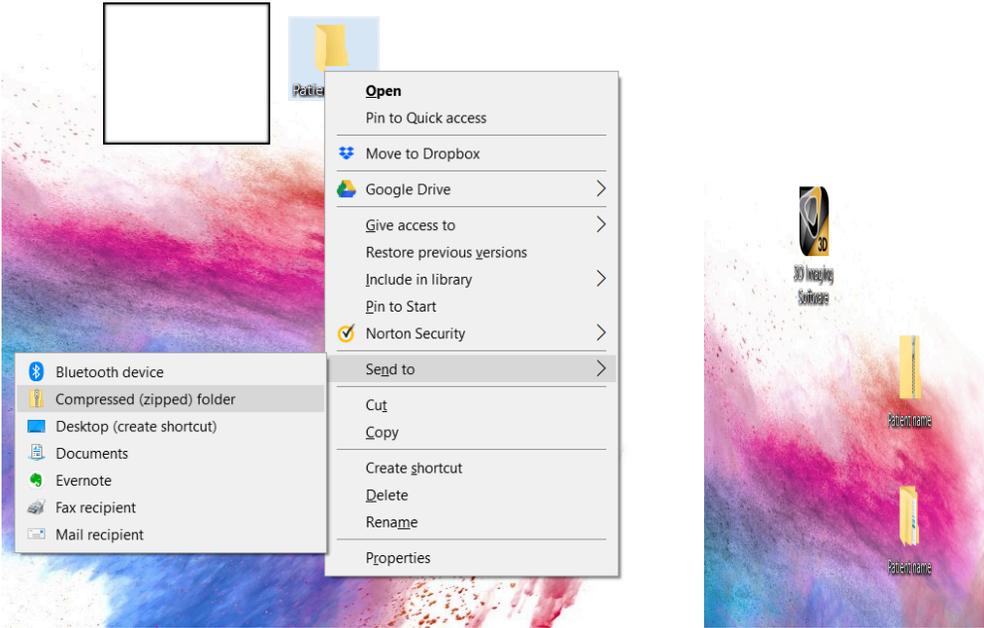

[Click here for instructions to create a case with mysecurepractice.](#)

[Click here for instructions to create a case with Hightail.](#)